



SPECIMEN REQUIREMENTS

For HAE Genetic Testing

Angioedema Genetic Testing	
<i>Use</i>	Angioedema genetic panels
<i>Specimen Type</i>	Whole blood
<i>Alt. Specimen Type</i>	Patient self-collected saliva according to the Zymo collection kit instructions (see table below)
<i>Container</i>	Lavender-top tube (contains EDTA)
<i>Volume</i>	5 ml
<i>Minimum Volume</i>	3 ml
<i>Handling</i>	Invert the lavender-top tube 8-10 times and store at 2-8°C immediately after collection. Ship the specimen with a cold pack (2-8°C) to the laboratory on the day of collection for next-day delivery.

Angioedema Genetic Testing - Alternative Specimen Type (Only if whole blood collection is not possible)	
<i>Use</i>	Angioedema genetic panels
<i>Specimen Type</i>	Saliva
<i>Container</i>	Zymo collection tube
<i>Volume</i>	4 ml
<i>Minimum Volume</i>	4 ml
<i>Handling</i>	Collect the saliva specimen according to the Zymo collection kit instructions and store the specimen at room temperature (15-25°C). Ship the specimen with a cold pack (2-8°C) to the laboratory on the day of collection for next-day delivery.

Additional Instructions

Please call (877) 888-2973 or email info@virantdx.com for questions and shipping assistance.

Please visit <https://virantdx.com/testing-solutions/> for our complete test menu and detailed test information.

Virant Diagnostics, Inc.

11002 Veirs Mill Rd, #404, Wheaton, MD 20902
Website: virantdx.com Email: info@virantdx.com
Phone: (877) 888-2973 Fax: 888-713-3456

Document ID: VDX-SPE-302-v2
Last modified: 10/04/2023