

Radaricesculptor

South Korean Radar Ice Carving Machine



Read the instructions before your use

User Instructions:

Dear User:

Thank you for choosing our equipment for better product results, the following are our sincere suggestions, please check:

- ★✦ Please read this manual carefully and keep it properly for future reference
- ★✦ Please install, use and operate the machine as required in this

instruction

- ★✦ Do not remove the machine or replace any parts of the machine
- ★✦ Other than authorized personnel, others open cap and open host is prohibited
- ★✦ Authority to Ban Repair and Refurbishment
- ★✦ Feel free to contact us for any problems occurring during the operation

1.1 What is a radar ice sculpture

Radar ice sculpture is a new kind of non-free technology, currently for facial muscle, body fat layer positioning, tightening and dissolved, high-tech beauty instruments. Radar ice sculpture instruments can effectively improve wrinkles, skin elasticity, shaping, and collagen formation. High intensity focused radar mechanical wave thermal energy up to 65 °C, can act directly on the skin SMAS layer (about 3.5mm~4.5mm), and will contract the skin muscle layer. In addition, it can lose weight by melting the fat layer 10mm~20mm depth. Special water circulation cooling mode, effectively protect the skin deep anti-aging, soothing sedation.

Enough penetration depth, strong tightening;

Painless, no rest vacation, no special care;

Simple operation, the process is about 40 minutes;

Safe and avoid cross-infection;

Long duration of the effect;

Ultra-high ROI;

1.2 Instrument principle

Using the good tissue penetration of mechanical radar wave, and the combination of focused ultrasonic beating principle, accurately transfer thermal energy to the SMAS layer of the skin (about 3.5mm~4.5mm), effectively stimulate fibroblast cells to divide new cells at 40 times the speed per second, dissolve facial fat cells, tighten skin contour, repair damaged fiber tissue, stimulate collagen cell regeneration, and quickly achieve the effect of thin face and compact Q bombs. Meanwhile, the radar wave precision

Positioning, can actively identify fat cells, through the unique probe friction treatment directly to the depths of the target, so that the patient will not feel pain. One treatment is equivalent to the effect of three ultrasound knives. Radar line carving instrument has very good treatment effect for law lines, Sichuan lines, apple muscle, acne face, hormone face, large face, large pores, meat face, cake face, baby fat, cheek proud flesh, double chin and other problems

1.3 Instrument action

Rapid wrinkle removal: can effectively remove fish's tail lines, head-up lines, perioral lines, lip lines, legal lines, puppet lines, etc., as well as up and down the neck, hands and body wrinkles.

Tight shaping: a single time can improve the facial skin sagging, double chin, fuzzy jaw curve, and drooping eyes and other problems.

Fiber slimming: for the postpartum figure fat, loose skin, butterfly sleeves, loose back waist and waist and other problems quickly solved. Tighten the face to relax the skin and control the energy emission interval without pain.

Hot and cold induction hammer: lift to relieve the redness, cold compress-10 °C can instantly shrink the pores, lock the moisture and nutrients. You can also use a warm hammer to act the nutrients on the skin to promote absorption and rejuvenation.

1.4 Product Advantages

1. Depth advantage: directly on deep direct fascia (smAs), muscle and fat.
2. Thermal hottest: non-invasive stimulation of collagen and elastic fiber regeneration.
3. Accurate radar: focused supersonic wave positioning system, precise treatment will not hurt the surrounding organization.
- 4, The effect is immediate: the tension can be owned.
5. No recovery period: beautiful whole body, completely does not affect the life and work and rest.
6. Ice feeling is comfortable: relieve muscle pain, reduce skin fatigue, redness and swelling, and resist the repeated growth of acne.
7. No surgery, no wounds, and say goodbye to the micro-whole power.
- 8, lasting: You will have tender skin profiles for a long time.
9. Process comfort: energy across the skin, the skin will not have a little pain.
- 10, the course of treatment effect: bloom confidence light, rewrite the fate of aging.

2.1 Instrument preparation Verify that all the items of the instrument are complete before starting the operation



Note: Select the treatment head according to the customer's skin condition

Anti probe: enhance face profile and fade texture

Hot and cold induction probe: soothe and calm the skin and promote absorption

2.2 Access and cutting method of instruments

When the ★ power supply is connected to the power system

Insert the power plug into the socket and press the main power switch [-] to start the instrument. When the ★ cuts off the power supply

Press the main power switch to the [O] position and unplug the power off.



PRODUCT BRAND	HC-FASHION	THE NAME OF THE INSTRUMENT	Radar ice sculptures
TYPE	Noninvasive carved	POWER	220V,60HZ
MODE OF OPERATION	Touch screen operation	ENERGY GEAR	5-speed
RADIO FREQUENCY (RF) TIME	0.5-0.9°C	RADIO FREQUENCY INTERVAL	0.5至2.0
LANGUAGE	English	POWER SUPPLY MODE	plug-in
HAMMER ICE PATTERN	Temperature can reach -10°C	HOT HAMMER MODE	36/39/42 °C
PRODUCT EFFICACY	Lift tight skin and relieve	APPLICABLE SCENARIO	Home/Business



3.1 Treatment site

Treatment areas are restricted areas, sensitive and comfort areas. Red area, eyebrow area; wide parotid area at the front of the ear; and wide area beside the corners of the mouth. Blue area of sensitive area of trachea and thyroid area, area marked by touch, frontal area, cheekatic area and mandible. Comfort zone green area, cheek, mandible commonly known as double jaw and neck broad muscle anterior muscle bundle.

3.2 Operation Suggestions

Instrument test parameters set, probe disinfection, right hand operation, left hand comfort lift, probe perpendicular to the face sliding, facial order, from bottom up, from the middle to both sides, grasp the rhythm, aging and serious area can focus on operation many times, after the whole face operation, once a week after a mask, to add a lot of water to the skin. Avoid eating spicy and stimulating seafood, beef and mutton wine.

3.3 Notes

Stop the same part for more than 5 seconds.

Forehead, thyroid, ear to lower Hubei and other nerve areas prohibited

At the operation area of about 5 mm in 1 second, move the probe.

No prohibited on objects outside the skin.

Dental or around the teeth, implant teeth, there may be cold teeth. You can inhale to drum your mouth or wrap your teeth with gauze.

Around the eyes, lower Hubei, lips and other nerves through the places, please do not operate.

The thyroid area around the neck is not operable.

When treating obesity, where the nerve passes, where the blood vessels pass, please do not operate.

3.4 Taboo population

Patients with acute diseases, patients with bleeding diseases above 38 °C.

People with hypertension, diabetes, infectious diseases, and patients with tuberculosis diseases.

Patients with malignant tumors, heart disease patients, and pregnant women.

Allergic reactions, hemophilia patients.

Chronic use of steroid hormone agents and liver dysfunction due to telangiectasatation.

Patients with primary explosion disease, patients with confirmed infectious diseases.

Patients with facial black skin disease, long-term hospital visits, body fever.4.

Operating method

Step: clean the face, wipe the saline, can massage the part, relax the skin.

Step: Apply the medical gel evenly on the operating site

Step: open the radar carving instrument to start operation, note (radar ice carving instrument has two probes, can act on the face and body) use the corresponding probe to start in the skin, with the pull beauty technique to pull the skin drooping part up. Each site operation time varies, depending

to the skin.

Step: Import beauty products into the skin after the above steps

5. Warranty Services

★ can accept free warranty for faults in normal use within 1 year

1. Free warranty only applies to the instrument body

2. Items than warranty during free warranty period

★ cannot enjoy free services without complying with the instrument startup steps and safety and management matters as stated in this book, and please pay special attention to the matters

Exclude free warranty when pulling adapter

Users use instruments or modifications for other uses, excluded from free warranty period when disassembly

(The Company reserves the right of final interpretation)

6. Paid warranty

One year after purchasing the instruments, even if the instrument is damaged due to consumer negligence or other natural disasters (fire, flood, impact, earthquake, lightning strike, etc.)

Paid guarantee is as follows: damaged instrument caused by the modification of self-decomposition instruments or inattention to the use (water inlet, damage, etc.)

Product Guarantee Letter

Model Number

Product Number

Product Name

Purchase Date

Date Date

Company Address

Mail address

Name:

Warranty Time

One year

Notes

* Be sure to provide this guarantee when making free repairs

* Consumer failure occurs under the category of normal use, and the manufacturer provides free maintenance services within the maintenance period

* The product is faulty during service but not with a faulty power cord and improperly used by the operator

* Any disassembly instruments, independent dismantling and modification of internal structure, caused by artificial fall or beating

For the fault, the manufacturer is entitled not to repair / charge related parts.

* Product guarantee will not be provided in two copies, so keep