

Foam Wedge Backrest Insert - S40

INSTALLATION GUIDE

Adhere two strips of velcro onto upper part of the velcro on the Carbon Fiber Seat back, and two strips of velcro onto the upper part of the velcro on the cushion



Pull backing off of the velcro on the seat back and carefully push the Foam Wedge onto the seat. Next, remove backing from velcro on cushion and push cushion onto the Foam Wedge.





Allow velcro adhesive to cure for 24 hours before attempting to move wedge or cushion. Once the velcro is cured, the wedge can be moved up and down the seat for support wherever you most need it. You're all set. Enjoy!