

Available on Google Play soon!

Until then, please use the following instructions:

Installing the Vesticam v6.2a on your Android Device*

1. Connect the Android to WiFi (or use SIM to connect to the internet)
2. Use email or sms to send this link to your android device
https://drive.google.com/file/d/1cYz_Ma_TrVjLGcQttrq5WnbFjZuv8TWR/view?usp=sharing
3. Open the link to install vesticam-6.2.apk with **“Package installer”**and wait
4. If prompted, **“agree”** to Google Drive access.
5. Once installed, **select “Done”**
6. Move/copy the Vesticam App icon to the Home Screen.
7. Connect the Vesticam Camera to the USB port, open the App and grant access.
8. The Camera should preview. You can now view eye movement, and record and replay patient eye movement videos if you wish.

* The “OTG” feature is required to run external USB devices like the Vesticam external USB camera. Not all Androids have the “OTG” feature.

You can check if your Android has the OTG feature at www.gsmarena.com under “COMMS”... “USB”

If OTG is not a listed feature, install the Vesticam App 6.2 and check if OTG is available.

If you prefer, book a time and we can assist you to install the Vesticam App v6.2 on your Android device.