


TM-502 Body Shaping System Manual



Body Shaping System, has 10 paths pulse ports which can connect 20 pieces of electrode pad to different areas of body, with 6 unique function key and 12 groups set computer programs inside. No matter reduce fatness, slimming, exercise breast, the beauticians now can operate it free to meet the customers different demands. If to the customer with special demands, beauticians also can set the computer programs over 100 types by own to operate those special demands.

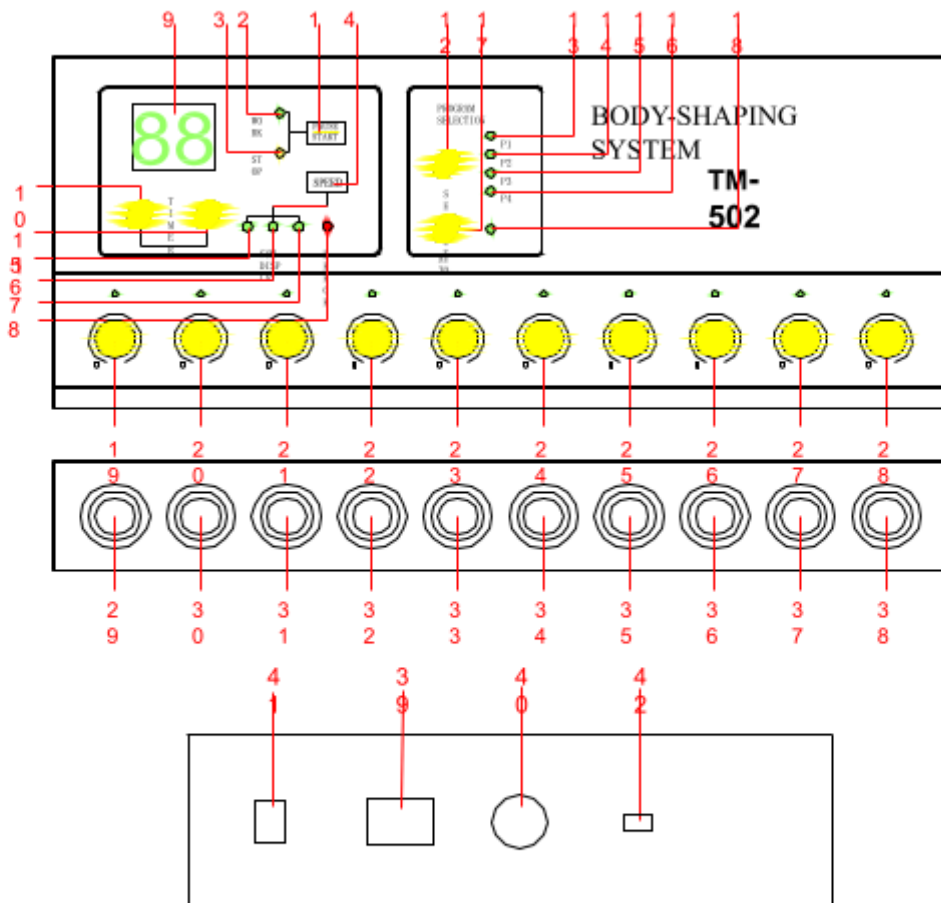

According to operating demands, to adjust timer to a right work time. The max work


Panel Description:

- | | |
|---------------------------------------|-------------------------------|
| 1: Start/Pause | 2,3: Work / Stop Indicator |
| 4: Speed | 5,6,7: Speed Shown (3 Models) |
| 8: Error Warning Indicator | 9: Timer Display |
| 10,11: Time Up / Down | 12: Program Select |
| 13-16: Program Shown (P1; P2; P3; P4) | 17: Auto Program |
| 18: Auto Indicator | 19-28: Min—Max Adjuster |
| 29-38: Output Port | 39: Cable Input |
| 40: Fuse | 41: Power ON/OFF |
| 42: 220/110V Change Button | |

Operation method

Plug in power supply, switch the 110V/220V power selection button to the 110V position, switch on the power supply on the back panel, and there will be time display on the instrument panel and the program is set to be 30 minutes.

If necessary, press the time increase or time decrease buttons 10 and 11 on the panel to set the operation time (If not all the power modulation groups are put in the zero position, the instrument will sound alarm. The instrument will stop the alarm 5 minutes after all the power modulations are put in the zero position). The maximum operation time is 60 minutes.

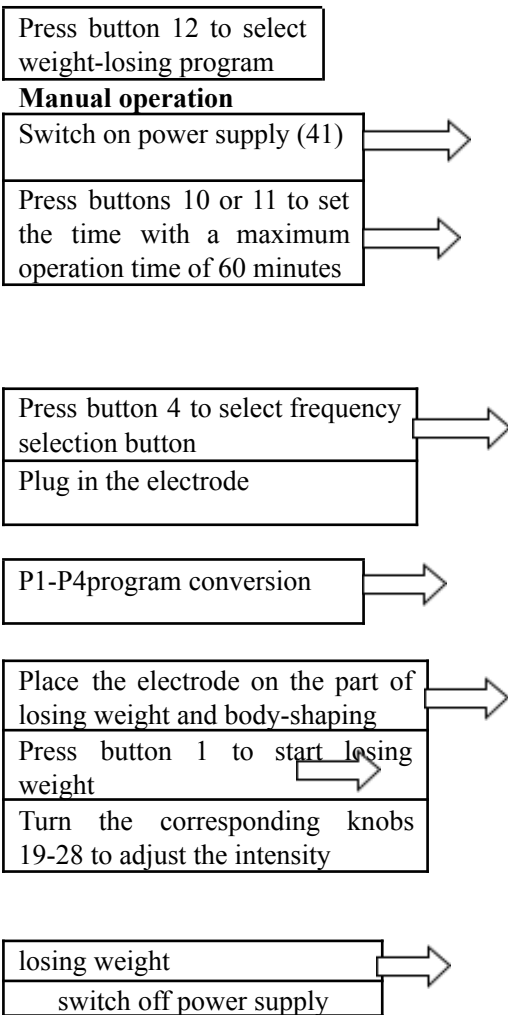

Automatic operation

Press buttons 10 or 11 to set the time with a maximum operation time of 60 minutes	→
Plug in the electrode	

Switch on power supply (41)	→
-----------------------------	---

Place the electrode on the part of losing weight and body-shaping	→	
Press button 1 to start losing weight		→
Press button 17 to start automatic mode		

Turn the corresponding knobs 19-28 to adjust the intensity	→
losing weight	
switch off power supply and take off the electrode	



Program Description:

P1: to fit muscle release and remove muscle ache and pain, work as skin massage.

P2: to fit breast area, muscle on legs and arms string, and buttocks and udders upgrade.

P3: to fit bionic movement (string drooping muscle, burn extra fatness).

P4: to fit reduce fatness, remove extra fatness cells through high intensional massage operation, to get slimming body.

Note: when to choose one program, the microcomputer inside machine will auto adjust the speed of operating to get best effect according to set program.

Attention

According to the size of the area of losing weight, select different electrode pieces. Basically the electrode piece should be used in close contact with the skin with big electrode piece for large area and small one for small area.

Before use, select dedicated weight-losing conductive liquid or clean water (instead of oily liquid) to apply to the surface of the skin and the electrode piece so that they become humid. While using bandage, make sure the electrode piece should come in good contact with the skin. Dry surface and ill contact with the surface may sting the user, a necessary point for the operator to remember.

Do not put the two electrodes near the heart in case the signal should pass through the heart and lead to discomfort.

For the part to be operated, heat with vapor for about 10 minutes, apply dedicated lotion and massage for 5-10 minutes, wipe the skin clean with wet towel or wet napkin, adjust the intensity button to increase the current

intensity gradually until the user can feel slightly the voltage under the sensory piece. While operating the instrument, the current intensity should be kept within the range acceptable to the user.

Do not use it for pregnant women or people with metallic conductor inside as well as people with acute suppurative inflammation.

It is suggested that application to the face should be as careful as possible (try to avoid the use).

Technical parameters

Program	Electric current (MA)	Frequency range (HZ)
P1	0.01-3	0.5-4
P2	0.015-10	0.8-5
P3	0.03-10	1-7
P4	0.05-11	1.2-8


Trouble-shooting

Troubles	Trouble-shooting methods
There is no current through the electrode piece	Ill connection with the output
The current loses control over its size.	Replace the current instrument


Specification:

Volt.	Power	Hz.	Temp.	G.W.	Size
220/110V	40W	50/60Hz.	5-40°C	5.8kg	47*39*22cm


Sketch map for electrotherapeutic acupuncture points

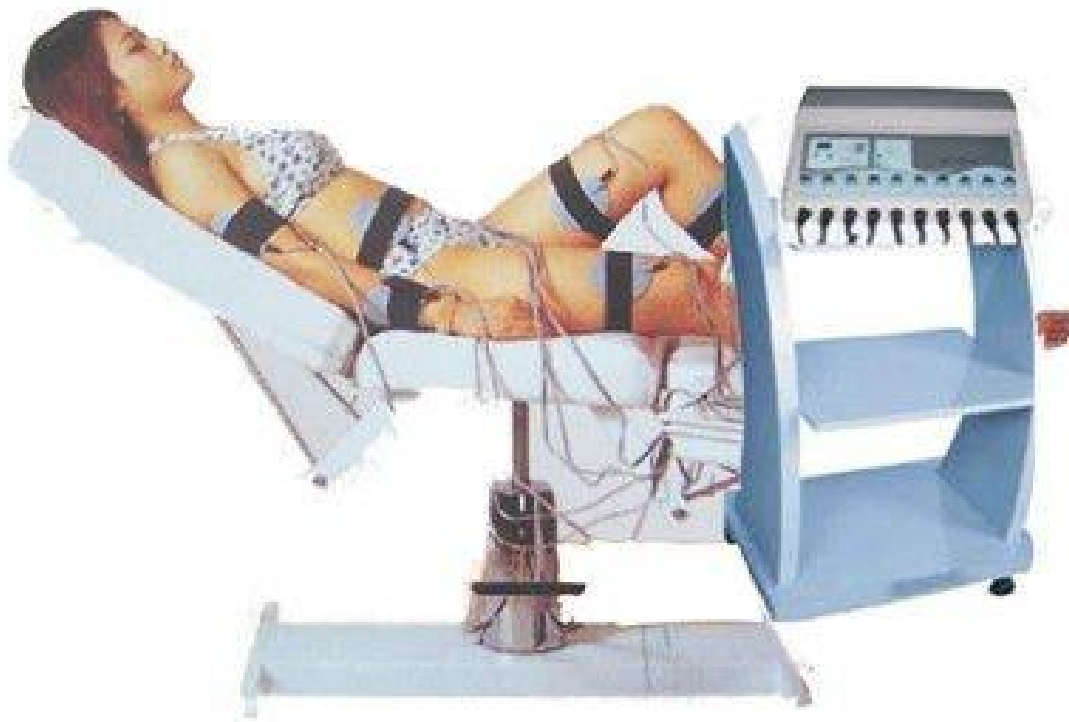


Reference map for the insertion of sensory pieces for losing weight and slimming body
(The pictures with the same number are of the same group)

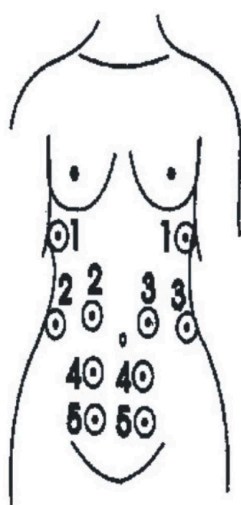

Insertion map for sensory pieces which are mainly for the stomach, tightening the muscles and losing weight

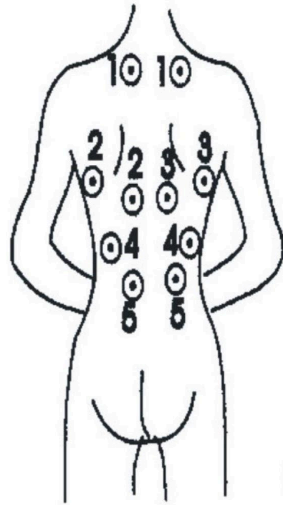


Insertion map for sensory pieces which are used to tighten the lower and upper part of the back to consume fat

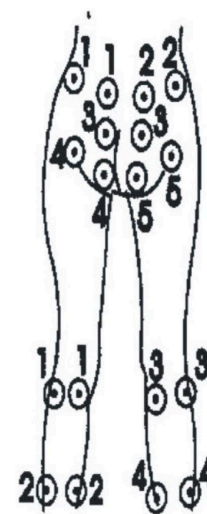


Insertion map for sensory pieces which are used to tighten the muscles in the arms and the crux to consume fat



Insertion map for sensory pieces which are used mainly to tighten the muscles in the arms to consume fat

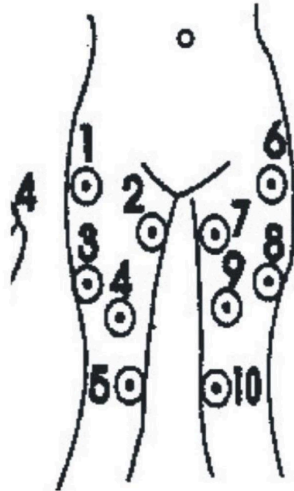

Insertion map for sensory pieces which are used to consume fat in the front side of the lap and tighten muscles

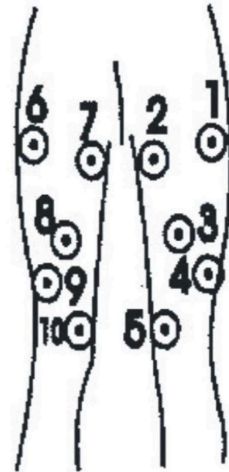

Insertion map for sensory pieces which are used to consume fat in the back side of the lap and tighten muscles