

# TableOnCloud Tips

The University of Michigan's Anatomage TableOnCloud subscription is for U-M affiliates only. Please send questions to [thl-anatomage-help@umich.edu](mailto:thl-anatomage-help@umich.edu)

**Important:** Login to TableOnCloud **ONLY** during your scheduled reservation time. Do not login before your reservation time starts or remain logged in to TableOnCloud after your reservation time has ended. TableOnCloud often has several reservations when class assignments are due so it is very important for users to respect the reserved time of other users.

## Tips:

- **Google Chrome Preferred**
  - Google Chrome is recommended for best performance
- **Software may take time to load**
  - Make sure your reservation time allows you to login and load the needed anatomy models/structures prior to the start of your class/group meeting
  - For example, if your class/group meeting is from 11:00am - 12:00pm, make your reservation for 10:30am - 12:30pm and login by 10:45am to load anatomy models/structures in advance and give yourself a little extra time after your meeting in case your meeting runs late
- **Multiple Users for Classes/Group Projects:** Only **one user can interact** with TableOnCloud **at a time**. Multiple users (no more than 3) with the same TableOnCloud login credentials can be logged into TableOnCloud, but clear communication needs to be established to determine who is the Primary User and who is “driving” the software at all times.
  - **Primary User**
    - Log into TableOnCloud first: we recommend doing this at least **15 minutes before** your class/group meeting starts
    - Open model(s) and any required presets **prior** to other users joining the session
    - Serves as the primary “driver” of the software and clearly communicates when other specific users (one at a time) may interact with the tool
    - Log off TableOnCloud when the session has ended
  - **All Users**
    - Refrain from interacting with TableOnCloud when another user is driving the session



- **If leaving the session early**, just close the window. **Do not log out**, as it will log out the entire group.
- **Mouse/Trackpad + Keyboard Controls:** To interact with TableOnCloud, both the mouse/trackpad and keyboard need to be used at the same time for some functions. *Only use one finger to click if using a trackpad.*

Comparison of TableOnCloud controls vs. the Anatomage Table:

Controls	TableOnCloud (Keyboard + Mouse)	Anatomage Table (Touch Gesture)
Rotate	Left Click (hold)	One finger (drag)
Pan	Shift + Left Click (hold)	Two finger (drag)
Zoom	Ctrl + Left Click (hold)	Two finger (pinch)
Spin	Space + Left Click (hold)	Two finger (pivot)

- **Maximum TableOnCloud Session Duration:** 2 hours. TableOnCloud will disconnect after 2 hours of use. Users can log back in if additional time is needed during your scheduled reservation time.
- **Maximum Idle duration: 15 minutes:** A pop up warning message will appear after 15 minutes of inactivity asking if you are still using the software. If you do not respond to this message, TableOnCloud will close.
- **Up to two software sessions** can be loaded at a time
- **If a user has logged into TableOnCloud on an iPad**, they must be sure not to touch the surface of the iPad while another user is driving.

\* **Mac Users Note:** Do not use Command +/- to zoom in and out. A magnifier icon and an additional cursor will appear and zooming will be difficult. Pressing **Control** on the keyboard while left clicking the mouse and moving the mouse forward and backward is an easier way to zoom.



Revised 6/2024